IFU HPMDC Rev. 08/2022-05-24

## Instruction for Use Disposable Tourniquet cuffs

*For better care!*

**DEUTSCH • ENGLISH • ESPAÑOL • FRANÇAIS • ITALIANO • PORTUGUÊS  
SVENSKA • SUOMI • NORSK**

**Symbol definition:**

The following symbols may appear on the device packaging or in this Instruction For Use (IFU)

Symbol	Definition
	Caution
	Manufacturer
	Production Batch code
	Reference number
	Consult instruction for use
	Medical Device
	Use by date, provided as year and month according the format YYYY-MM-DD
	Indication of conformity with essential health and safety requirements set out in European Directives
	Keep away from sunlight
	Keep dry
	Do not re-use
	Do not use if package is damaged
	Sterilized using ethylene oxide
	Number of pieces in carton

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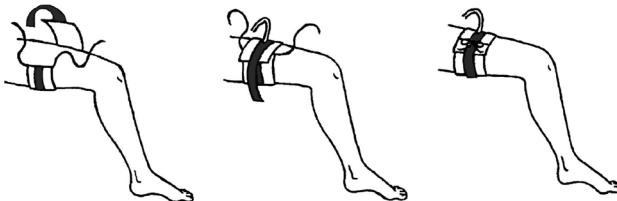
## Manual for the Hpm™ disposable cuff

### INDICATIONS

Tourniquet Cuffs are indicated for use in surgical procedures that require the interruption of blood flow and clear visualization of the operative field.

### APPLYING OF CUFF

Apply padding first as smoothly as possible to prevent skin trauma. Then apply the cuff as smoothly as possible to avoid wrinkles. See it to that hose(s) is directed proximally. Thigh or upper arm is preferred because larger amounts of tissue and muscle protect nerves from possible trauma. In obese patients, draw the skin and subcutaneous tissue distally during application. This helps to prevent distal slippage with subsequent loss of occlusion and loss of haemostasis. Hammarplast Medical offers many different sizes of cuffs.

### PRECAUTIONS

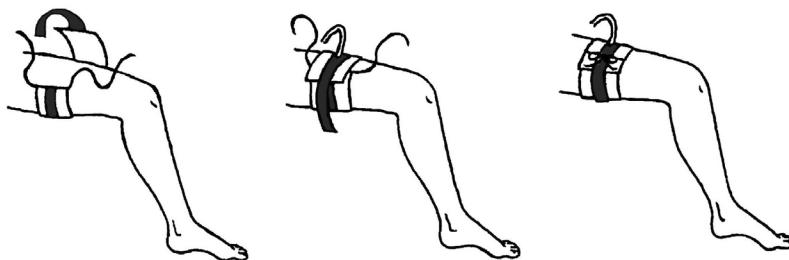
- Extreme care should be exercised in application of usage. All final decisions regarding tourniquet use are the responsibility of the operating surgeon.
- Exercise extreme care when using tourniquets of any type. Minimum pressure and application time should be used in all procedures. Preoperative personnel should be aware of the most common complications and prevention strategies.
- Carefully monitor set inflation pressure during the entire procedure to help ensure the desired pressure is being accurately maintained.
- Ensure that no caustic solutions are allowed to run under the cuff to avoid chemical burns to the patient. The total time of the operation should not exceed 2 hours to prevent nerve, muscle and tissue trauma. For planned long operations use a double cuff and switch between proximal and distal side frequently.
- The Disposable Cuffs are designed for single use. Reprocessing or resterilization is not recommended.

### ADVERSE EFFECTS

Vascular complications, neuromuscular or neurological injuries, tourniquet pain, ischemia, venous emboli or thromboembolism, blood vessel trauma, reperfusion problems and arterial occlusion have been reported with tourniquet use.

### INSTRUCTIONS FOR USE

1. The tourniquet should be applied as far proximally as possible, on either the upper or lower extremity, at the maximum circumference of the limb.
2. Select a cuff large enough to have a suggested three- to four –inch (7.5 to 10cm) overlap and avoid rotary movement after application.
3. Prior to application, wrap the area with a stockinette sleeve.
4. Avoid any wrinkles in the stockinette sleeve or cuff, secure the pressure –sensitive closure and tie.
5. Securely connect (single port or dual port) Inflation line to inflation system.

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Hammarplast Medical AB